



Improving Care. Improving Business.®

FOAM ELBOW IMMOBILIZERS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE


The DeRoyal® Elbow Immobilizer is intended to be prescribed by a properly licensed practitioner to help prevent patients from disturbing medical treatment.

CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNING

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this device must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this device to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by a medical professional.
- Inspect product for damaged or missing components before use.
- Discontinue use and consult your prescriber if the device or its components break or become damaged.
- Inspect the fit of the device on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if the patient experiences sensation changes, unusual reactions, swelling or increased pain while using this device. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

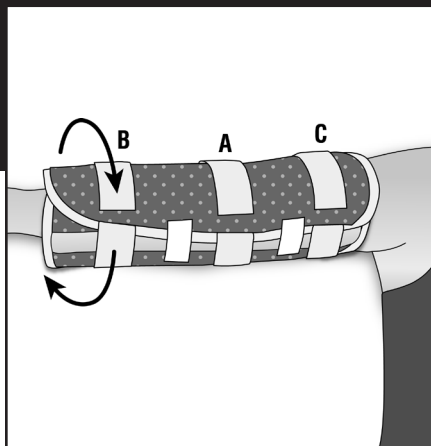

INSTRUCTIONS FOR USE

ADULT

1. Place the immobilizer under the patient's arm with the widest part toward the axilla.
2. The elbow should be centered between the two rigid bars.
3. Wrap the immobilizer around the arm starting with the side without straps.
4. Hold the immobilizer in place using the extra unattached hook tabs.
5. Secure the immobilizer by wrapping the center strap first, followed by the bottom strap and then the top strap. **NOTE:** Do not over-tighten, one finger should fit between the device and the patient to ensure circulation is not compromised.

PEDIATRIC

1. Wrap the immobilizer under the patient's arm. The elbow should be centered between the two stays.
2. Secure the immobilizer by securing the center hook tab first, followed by the bottom and top hook tabs. **NOTE:** Do not over-tighten, one fingers should fit between the device and the patient to ensure circulation is not compromised.

CLEANING AND/OR MAINTENANCE

Hand wash in lukewarm water with a mild detergent. Dry completely prior to wearing.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business®



Improving Care. Improving Business®



Manufacturer:

DeRoyal Industries, Inc.

200 DeBusk Lane

Powell, TN 37849 USA

888.938.7828 or (001) 865.938.7828

www.deroyal.com

Part#0-703 Revised 3/2019

©2019 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.