



Improving Care. Improving Business.®

ARM SLING

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Arm Sling is used for cast support, injuries to the arm, wrist or hand, acromioclavicular sprains, and anterior glenohumeral dislocations.


CAUTIONS

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber. Use only as directed.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the shoulder (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Wear clothing underneath this brace.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.



INSTRUCTIONS FOR USE

1. Attach shoulder pad by inserting strap through openings (if applicable). Slide strap to desired position so that the foam sits comfortably on the patient's shoulder.
2. Place affected arm in sling so that the arm rests in the fold of the sling against. Patients elbow should be pressed back snugly fitting in the sling.
3. Place thumb in thumb loop to prevent wrist drop (if applicable).
4. a.) Bring strap behind patient neck and over opposite shoulder. If hook and loop closure - bring strap through the O-ring(s), adjust to proper height and secure b.) If buckle closure, bring strap through buckle and pull the strap to adjust to proper height.
5. When applicable, use sliding buckle in back to adjust for proper fit.
6. Make sure sling supports arm securely.

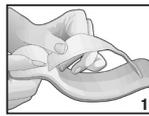

NOTE: If the arm sling is fit properly, the pouch should extend to the first joint on the little finger (the P.I.P. Joint). The arm sling should also be slightly elevated to help prevent edema.

CARE INSTRUCTIONS (IF APPLICABLE)

Hand wash in lukewarm water with a mild detergent. Air dry completely prior to wearing.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT



Improving Care. Improving Business.®

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



Improving Care. Improving Business.®



Manufactured by:

DeRoyal Industries, Inc.

200 DeBusk Lane

Powell, TN 37849 USA

888.938.7828 or (001) 865.938.7828

www.deroyal.com

Part#0-400 Revised 5/2020

©2020 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.