




Short Thumb Spica

Single Patient Use Only

Rx Only

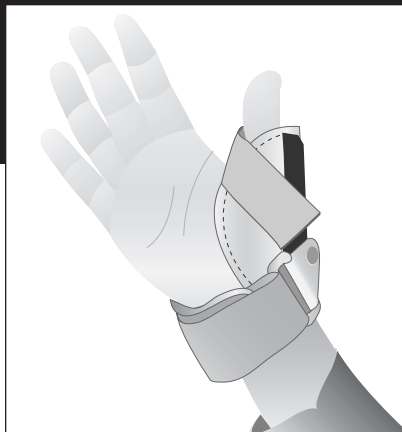

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use.

INTENDED USE


Post-thumb MCP fracture, ligamentous injury to MCP joint, collateral ligament injury such as Gamekeeper's thumb and CMC joint pain.

PRECAUTIONS

Consult your clinician immediately if you experience sensation changes, unusual reaction, swelling or increased pain while using this product.

INSTRUCTIONS FOR USE

1. Loosen the palmar strap but do not pull the strap out of the D-ring.
2. Slide the thumb into the thumb holster.
3. Secure the strap around the palm of the hand by attaching the strap to the hook material at the base of the splint.

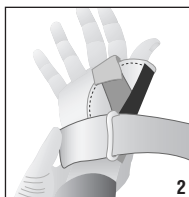

4. Secure the strap around the thumb.
5. Readjust straps for desired fit. Do not over tighten straps.



SIZING INFORMATION

Measure the width of hand at MP joint in inches.

Small	2 1/4" – 2 3/4"
Medium	2 3/4" – 3 1/4"
Large	3 1/4" – 3 3/4"

WASHING INSTRUCTIONS

Wipe with non-abrasive cleaner



Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Part #1-347
©2003 DeRoyal. All Rights Reserved.
Revision 02 Revised 9/03



Short Thumb Spica

Single Patient Use Only

Rx Only



IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use.

INTENDED USE



Post-thumb MCP fracture, ligamentous injury to MCP joint, collateral ligament injury such as Gamekeeper's thumb and CMC joint pain.

PRECAUTIONS

Consult your clinician immediately if you experience sensation changes, unusual reaction, swelling or increased pain while using this product.

INSTRUCTIONS FOR USE

1. Loosen the palmar strap but do not pull the strap out of the D-ring.
2. Slide the thumb into the thumb holster.
3. Secure the strap around the palm of the hand by attaching the strap to the hook material at the base of the splint.

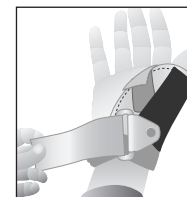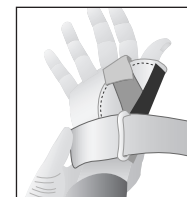

4. Secure the strap around the thumb.
5. Readjust straps for desired fit. Do not over tighten straps.



SIZING INFORMATION

Measure the width of hand at MP joint in inches.

Small	2 1/4" – 2 3/4"
Medium	2 3/4" – 3 1/4"
Large	3 1/4" – 3 3/4"

WASHING INSTRUCTIONS

Wipe with non-abrasive cleaner



Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Part #1-347
©2003 DeRoyal. All Rights Reserved.
Revision 02 Revised 9/03