


HIGH POWERED AUTOLYTIC DEBRIDEMENT

ALGIDEX Ag[®] WET GAUZE DRESSING


ALGIDEX Ag[®]
WET GAUZE

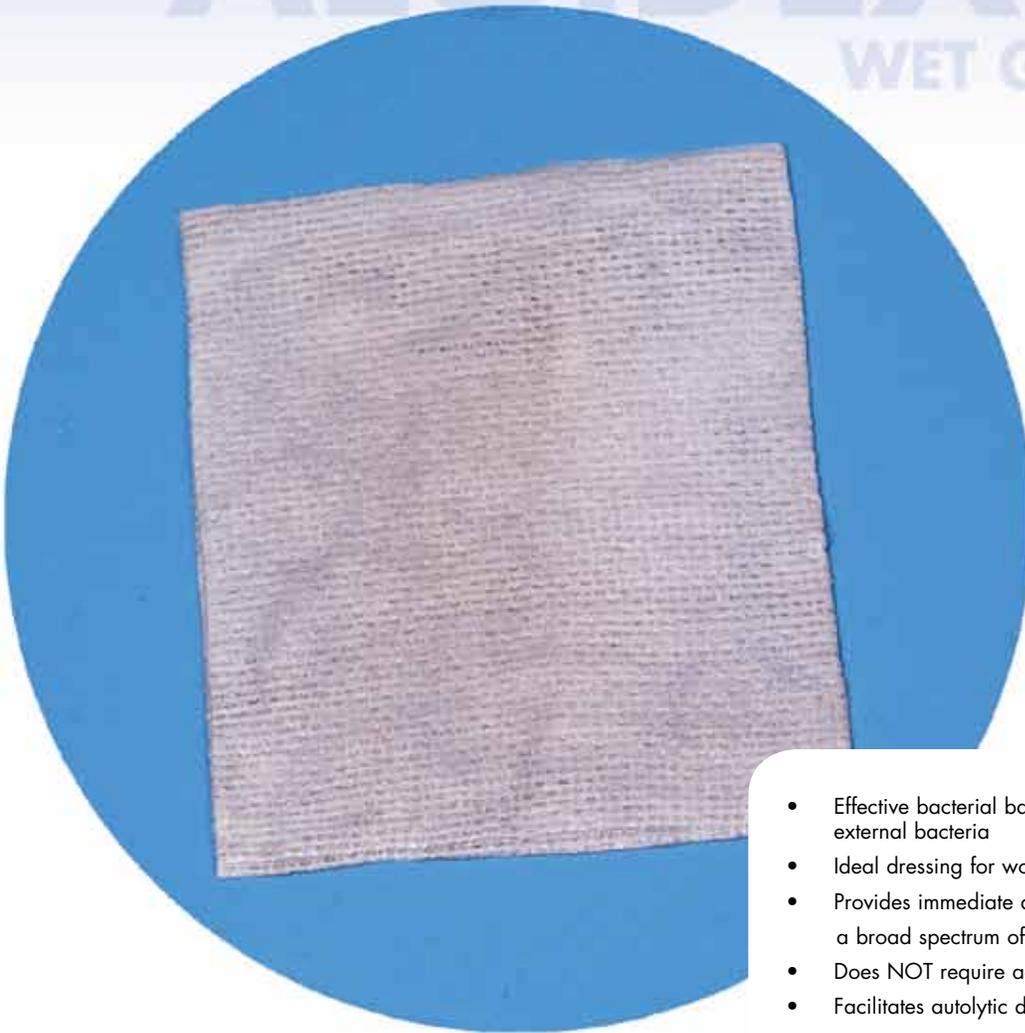

Unique combination of ionic silver, alginate, and maltodextrin provided in a **hyper-saturated non-woven gauze**.

- Effective bacterial barrier to help prevent contamination from external bacteria
- Ideal dressing for wounds with necrotic tissue
- Provides immediate and sustained antimicrobial activity against a broad spectrum of pathogens
- Does NOT require activation
- Facilitates autolytic debridement

 DeRoyal[®]

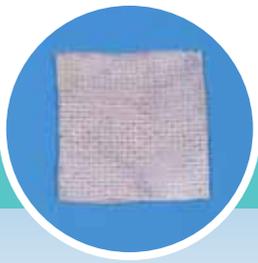

HIGH POWERED AUTOLYTIC DEBRIDEMENT

ALGIDEX Ag[®] WET GAUZE DRESSING

ANTIMICROBIAL ANALYSIS

Algidex Ag is a technologically advanced wound dressing that has been uniquely formulated with a combination of ionic silver, alginate, and maltodextrin to provide immediate and sustained antimicrobial activity against a broad spectrum of pathogens without inducing bacterial resistance. Algidex Ag has been clinically documented as a cost-effective dressing that provides excellent wound management with a longer wear-time and fewer dressing changes. The silver in Algidex Ag is completely and evenly distributed over the dressing in the ionic state.

FEATURES & BENEFITS

- Reduces bacterial colonization
- Absorbs wound exudate
- Decreases wound odor
- Specially woven gauze will not ravel, lint, or fray
- Can be cut to desired length
- Releases active ionic silver for broad antimicrobial effectiveness
- **Facilitates autolytic debridement**

INDICATIONS

Algidex Ag Wet Gauze Dressings have been hyper-saturated with the Algidex formulation and is the perfect dressing for use on **necrotic wounds** to help facilitate **rapid autolytic debridement**. Algidex Ag Wet Gauze Dressings can also be used on:

- Minor burns
- Skin grafts
- Skin donor sites
- Dermal ulcers (e.g. leg ulcers, pressure ulcers)
- Diabetic ulcers
- Abdominal wounds
- Superficial wounds
- Lacerations, cuts, and abrasions

DIRECTIONS FOR USE

1. Apply gloves.
2. Thoroughly cleanse wound.
3. Remove Algidex Ag Wet Gauze Dressing from packaging.
4. Place Algidex Ag Wet Gauze Dressing over shallow wounds or pack loosely into deep wounds.
5. Secure dressing in place with appropriate secondary dressing such as gauze, hydrocolloid, foam, composite, or film dressing. Secondary dressing selection should be based on the amount of wound drainage.
5. Dressing may be worn up to 7 days or until secondary dressing is required to be changed or reaches saturation.



REMOVAL

1. Apply gloves.
2. Remove secondary dressing.
3. Gently remove the Algidex Ag Wet Gauze Dressing.

Product #	Size	Qty/Cs
46-GZ22	2" x 2"	100
46-GZ44	4" x 4"	100



DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, Tennessee 37849
www.deroyal.com

TO PLACE AN ORDER OR FOR MORE INFORMATION
PLEASE CALL: **800-251-9864**
865.938.7828 (OUTSIDE USA)
800.327.0304 (FAX)

Reprint #0-2306
Rev. 4/12
©2012 DeRoyal