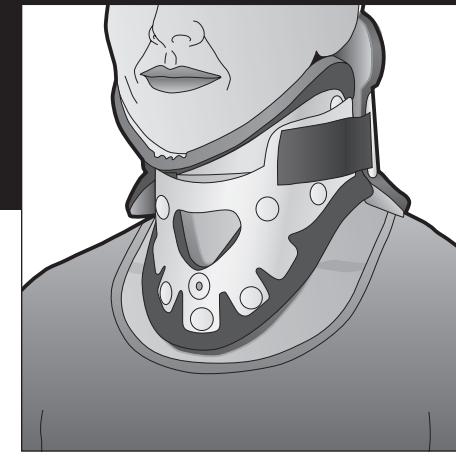
Improving Care. Improving Business.<sup>®</sup>**XTW EXTENDED WEAR COLLAR****COLLIER XTW EXTENDED WEAR  
(POUR UN PORT PROLONGÉ)****COLLARÍN XTW PARA USO PROLONGADO****XTW HWS-SCHIENE FÜR  
LANGZEITANWENDUNG****COLLARE XTW PER USO PROLUNGATO****COLAR XTW PARA USO PROLONGADO****XTW-KRAAG VOOR LANGDURIG GEBRUIK****XTW-KRAGE FÖR LÅNGTIDANVÄNDNING****XTW UZUN SÜRELİ KULLANIM İÇİN  
BOYUNLUK****XTW ΚΟΛΑΡΟ ΠΑΡΑΤΕΤΑΜΕΝΗΣ ΧΡΗΣΗΣ**

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NON STERILE

NOT MADE WITH NATURAL RUBBER LATEX

FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE  
TO SALE OR USE BY OR ON THE ORDER  
OF A PHYSICIAN OR PROPERLY LICENSED  
PRACTITIONER.**IMPORTANT INFORMATION**

Please read all warning and instructions before use.  
**WARNING** Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

**INTENDED USE**

The XTW Extended Wear Collar is intended to restrict flexion, extension, lateral bending and axial rotation promoting patient recovery.

**ADULT SIZING**

Circumference all are 13"- 21"  
**COLLAR CHIN DIMENSION**  
X-Short ≤ 1/4"  
Short 1/4"- 5/8"  
Regular 5/8"- 1 1/4"  
Tall 1 1/4"- 1 7/8"  
X-Tall ≥ 1 7/8"

**CHILD AND INFANT SIZING**

COLLAR	AGE	CIRCUMFERENCE
Infant Short	1 - 18 months	8"- 12"
Infant Regular	9 - 24 months	11"- 13"
Tot	1 - 3 years	11 1/2"- 14"
Child Short	2 - 5 years	12"- 14 1/2"
Child Regular	3 - 6 years	13"- 16"

**CARE INSTRUCTIONS**

Hand wash three removable pads lukewarm water, mild detergent, air dry. Replacement pads are available.

**STORAGE AND TRANSPORT CONDITIONS**

**KEEP DRY**

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

**WARRANTY**

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business, are trademarks or registered trademarks of DeRoyal Industries, Inc.