## EN: SURGICAL SKIN MARKER

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## FR: MARQUEUR CUTANÉ CHIRURGICAL

## DE: OP-HAUTMARKER

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## NL: CHIRURGISCHE HUDMARKEERSTIFT

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## NO: KIRURGISK HUDMARKØR

**MANUFACTURED BY:**  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.derooyal.com

**MedEnvoy Global B.V.**  
Prinses Margrietplantsoen 33 – Suite 123  
2595 AM the Hague  
The Netherlands

## PART#0-1862G I REVISED 10/23

©2023 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.

## SURGICAL SKIN MARKER

	<b>SINGLE PATIENT USE</b>
	<b>MEDICAL DEVICE</b>
	<b>STERILIZED WITH ETHYLENE OXIDE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

## IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

## INTENDED USE

Surgical Skin Markers are intended for use by a properly licensed practitioner for marking on the skin in perioperative or other procedural settings.

## CONTRAINDICATIONS

- Not for ocular use.

## WARNINGS

- DO NOT** reuse. Product is single patient use.
- DO NOT** ingest. In case of accidental ingestion, **DO NOT** induce vomiting. Rinse mouth thoroughly with water and get immediate medical attention.
- Avoid contact with eyes. In case of accidental contact, flush eyes with copious amounts of water. Seek medical attention as needed.

## CAUTIONS

- May cause irritation in sensitized individuals. If irritation occurs, wash affected areas with soap and water. If irritation persists, seek medical attention.
- Use care if applying prep solution over skin marker ink. Ink may smear.
- This device is for single patient use only. It is not intended for disinfection and subsequent re-use, which may result in device failure and/or create the risk of contamination.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/or may lead to device failure, which could result in patient illness, injury or death.
- If ink is leaking or dried out, discard and replace.

## DIRECTIONS

- Remove sterile pen from packaging.
- Remove pen cap.
- Mark desired area on skin with skin marker according to facility protocol.
- Allow the marker to dry before applying prep solution (not included).
- Dispose of skin marker after use on individual patient.

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## STERILIZATION INFORMATION

**NON-STERILE PRODUCT:** DeRoyal intends that non-sterile products for use in sterile environments be further processed by our customers, including further packaging and/or sterilization. These products are to be sterilized by ethylene oxide, utilizing the following parameters: Temperature: 120°F - 135°F (48.9°C - 57.2°C), Ethylene Oxide Gas Concentration: 625 - 800 mg/l, Exposure Time: 4 hrs minimum (Preconditioning: Temperature 100°F - 120°F (37.8°C - 48.9°C), Humidity 55-75%, Time 18hrs minimum), (Aeration: Temperature 100°F - 135°F (37.8°C - 42.2°C), Time 18hrs minimum). Other sterilization methods have not been validated by DeRoyal and may damage the product which could cause a device malfunction and/or injury to the patient.

The device has not been evaluated for re-processing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/or may lead to device failure which could result in patient illness, injury or death.

## STORAGE CONDITIONS

Do not store product at extreme temperatures or in a moist/damp environment.

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

## WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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