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## WARRIOR® RECOVERY POST-OP KNEE BRACE

### RX ONLY SINGLE PATIENT USE NON-STERILE NOT MADE WITH NATURAL RUBBER LATEX

### IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

### INTENDED USE

The DeRoyal® Warrior® Recovery Post-Operative Knee Brace provides knee immobilization or range of motion limitations as prescribed by a medical practitioner. This brace may be used post-injury, postoperatively or throughout rehabilitation.

### CONTRAINDICATIONS

Patients with poor peripheral circulation, diabetes or decreased skin sensitivity should not use this product.

### PRECAUTIONS

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for duration of use. These instructions do not supersede hospital protocol and/or direct orders of patient's physician.
- Inspect brace for damaged or missing components prior to patient application.
- Inspect brace for proper fit immediately following patient application.
- Discontinue use if flexion/extension stop or drop lock does not engage at the proper angle.
- This knee brace is designed to help recovery and reduce the risk of injury only. It is not intended to prevent or eliminate knee injuries.
- Flexion and extension stop adjustments to the hinge should be made only by, or under the advice of, your physician.
- Use only as directed by your medical professional.

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### WARNINGS

- Discontinue use of the device and consult physician or properly licensed practitioner if it becomes damaged in any way, limiting its ability to perform as intended.
- Discontinue use and consult physician or properly licensed practitioner if brace does not support patient's weight, provide proper knee immobilization or limit range-of-motion.
- Discontinue use and consult physician or properly licensed practitioner if brace causes muscle contraction or soft tissue damage.

### INSTRUCTIONS FOR USE

#### BRACE APPLICATION

- Unhook all clip buckles.
- Align the hinge of the brace with the knee joint. Brace should fit 2" below groin and 2" above ankle bones.
- Fasten clip buckle just below the knee joint. Repeat with strap above the knee joint.
- Next, fasten the two remaining straps beginning below the knee then above the knee.
- Strap length can be adjusted by removing Y-tab closures on strap ends, trimming straps to desirable length, reapplying Y-tab closures to the strap ends and tightening all four straps. Straps should be snug but comfortable. Excess strapping can be trimmed before applying Y-tabs.

#### LENGTH ADJUSTMENT

- Unlock length adjustment levers located at the top and bottom on both sides of brace (blue levers).
- Adjust to accommodate leg length by sliding the telescoping bars. Brace should fit 2" below groin and 2" above ankle bones.
- Lock length adjustment levers securely and ensure each side is equal in length.

#### PAD PLACEMENT ADJUSTMENT

- The pads located below and above the range-of-motion (ROM) hinges can be moved or removed to accommodate incision sites or for desired comfort or fit.
- To move or remove these pads, unscrew the single hex screw slightly with a 3/32" or 2.5mm hex wrench and slide pad to the desired position.
- Once the desired position is achieved, retighten the hex screw with the hex wrench. **NOTE:** Do not overtighten.

#### FLEXION/EXTENSION ADJUSTMENT

- The range-of-motion (ROM) hinge can be set to accommodate the prescribed degree or degrees of flexion and extension of the knee brace.
- With the patient's leg and brace fully extended,

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set the flexion stop by pulling out on the stop face (white stop). Slide the stop to the desired degree of flexion and push stop face into place to lock the flexion stop.

- Once flexion is set, pull out on the extension stop face (white stop) and slide stop into desired degree of extension. Push stop face into place to lock the extension stop.
- Perform function on both ROM hinges.

#### DROP-LOCK ADJUSTMENT

- To quickly lock the ROM hinges to drop lock presets, first align the blue arrow on the face of the ROM hinge with the corresponding lock presets (0°, 15°, 30°, 45°) located on the hinge bar, then push the blue lever on the ROM hinge into the down position.
- To unlock, pull blue lever into the up (unlocked) position.

#### STRUT ADJUSTMENT

- The Warrior Recovery post-operative knee brace also features a bend zone built into the lower medial and lateral struts of the brace to accommodate patient anatomy. This zone is indicated by a circular hole in the strut located just below the ROM hinge.
- To bend, place the bend zone against the edge of a hard surface (table, desk, etc.) and apply gradual force to the lower strut while holding the ROM hinge and upper part of the brace steady until the appropriate amount of bend is achieved.

#### ADDITIONAL STRAP PADDING

- The Warrior Recovery post-operative knee brace also features additional strap padding to help prevent migration and enhance comfort of the device.
- To apply strap padding, determine appropriate length of strap padding, trim to desired length and apply on the interior of the applicable strap.

#### CLEANING INSTRUCTIONS

Hand wash with mild detergent in lukewarm water. Air dry.

#### WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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DeRoyal®

Improving Care. Improving Business.™

Manufacturer: DeRoyal Industries, Inc. 200 DeBusk Lane Powel#, TN 37849 USA 888.938.7828 or 865.938.7828 www.deroyal.com

ONET BV  
Hommelertweg 286  
6436 AM Arnhemrade  
The Netherlands

#### Part #0-1891G

Revised 6/2015