FOAM BED RAIL PADS

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FOAM BED RAIL PADS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX

IMPORTANT INFORMATION

 Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Foam Bed Rail Pads are intended to be used to help reduce the risk of bed entrapment, help prevent bed exit and help protect against side rail injuries.

CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.

 WARNINGS

- Improper application or use of device may result in serious injury or death.

CAUTIONS

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Inspect product for damaged or missing components before use.
- Discontinue use and consult your prescriber if the product or its components break or become damaged, or cleaning is no longer

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sufficient to sanitize product.

INSTRUCTIONS FOR USE

- Place the bed rail pads over each side rail.
- Secure the hook & loop closures around the side rails.

DIMENSIONS

60” L x 18” H (152 x 46 cm)

CLEANING AND/OR MAINTENANCE

Please properly dispose of the product per facility's policies and procedures.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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