
14. As needed, use the handles located along the patient and top of the sheet to position the patient.

APPLYING THE WEDGES

1. Take the two long handles and drape them over the patient towards the opposite side of the bed.
2. Hold onto both handles and pull to turn the patient on their side. If needed, utilize the short handles to hold this position.
3. Ensure the patient is completely side-lying and locate the sacrum.
4. Fully insert the first wedge above the sacrum to support the upper torso. Wedge handles may be used to aid in proper wedge positioning.
5. Fully insert the second wedge below the sacrum to support the hips, buttocks, and thighs.
6. Gently turn the patient onto their back (with wedges underneath). Readjust wedges per facility protocol and/or as needed.
7. Smooth out the sheet and pad, and drape handles over the edge of the bed. Ensure handles are draped away from the patient to prevent entanglement with patient and/or tubing. **NOTE:** Follow facility protocol for number of side rails to remain in the upright position while patient is unattended.
8. Return bed to the lowest height, or appropriate height as determined by care team and/or facility protocol.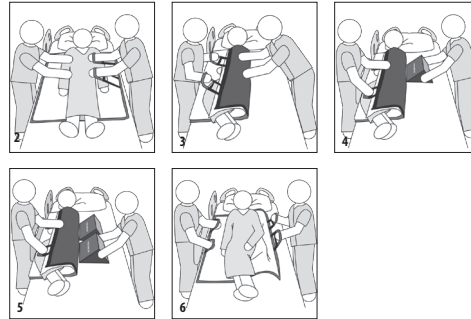

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from deroyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

CLEANING INSTRUCTIONS

Absorbent PAD - If the pad is soiled, dispose and replace promptly in accordance with facility protocols and guidelines.

Sheet - If the sheet is soiled, dispose and replace promptly in accordance with hospital protocols and guidelines. Do not launder.

Wedges - If the wipe able top and bottom surface (black surface) of the wedge is soiled, wipe down with a damp cloth or hospital-approved disinfectant. Dry completely prior to use. If the product is damaged or cleaning is insufficient, replace with a new one.

Replacement sheets, wedges, and pads are available.

SIZING

Universal Sheet (35" x 55")
X-Large Sheet (45" x 55")

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

PRUVENTOR TURNPRO® SYSTEM TURNING AND POSITIONING KIT

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® PRUventor TurnPRO® System is intended to assist and maintain proper positioning of the patient while helping prevent sacral pressure injuries.

The sheet is intended to provide assistance to the caregiver when repositioning the patient.

The wedges are intended to off-load the sacrum.

The absorbent pad is intended to help optimize the body's microclimate through breathable material and moisture-wicking technology.

CONTRAINDICATION

Patients who have an unstable spinal injury should not use this system, unless a decision to use has been made by the healthcare professional.

WARNING

- Prior to use, inspect the kit components for damaged, missing, or contaminated product.
- Consult physician or properly licensed practitioner immediately if the patient experiences sensation changes, unusual reactions, swelling or increased pain while using this device.
- **DO NOT** attempt to transfer, carry, or mechanically lift the patient while using this device.
- **DO NOT** hold onto the absorbent pad as a method to position the patient.
- Prior to positioning a patient, ensure movement will not disrupt treatment.
- Make sure patient's head does not drag during positioning.

CAUTIONS

- The components within this system have only been tested in conjunction with each other. Use of outside components could alter proper performance.
- Ensure the pad and sheet lay flat and smooth beneath the

patient.

- To prevent entanglement with patient and/or other medical devices, ensure handles are always draped away from the patient.
- Follow your facility's policies and procedures for frequency of patient repositioning and monitoring.

INSTRUCTIONS FOR USE

Follow your facility's policies and procedures for safe patient handling.

PREPARE THE BED

1. Ensure bed brakes are in the locked position.
2. Ensure the bed is as flat as possible, provided it does not interfere with the patient's condition.
3. Lower or raise the bed to ensure the top layer of the support surface (e.g. mattress) is at the clinician's waist level or elbows.

APPLYING THE SHEET W/ ABSORBENT PAD

1. Ensure side rail(s) are in the upright position.
2. Lower the side rail(s) on the side the clinician is standing to apply the sheet and pad.
3. Unfold the sheet and pad.
4. Keeping the pad directly on top of the sheet, tightly roll approximately half of the pad and sheet length wise, to ensure excess material can be tucked underneath the patient in Step 8.
5. Position the top of the rolled sheet and pad directly above the patient's shoulders.
6. Gently turn the patient away from the clinician on to their side and keep them side-lying. Follow facility's safe patient handling recommendations and/or protocol for number of individuals required to safely turn a patient.
7. Bring the rolled sheet and pad to the center of the bed.
8. Push and tuck the rolled portion of the sheet and pad underneath the patient, ensuring the top of the sheet sits directly above the patient's shoulders.
9. Raise the side rail(s), and gently turn the patient onto their back.
10. Lower the side rail(s) on the other side.
11. Gently turn the patient onto the exposed sheet and pad and keep them side-lying.
12. Unroll the rest of the sheet and pad. Drape the handles of the sheet over the edge of the bed.
13. Raise the side rail(s), and gently turn the patient onto their back.

Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART #0-2010 | REVISED 5/2026

©2026 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal, PRUventor TurnPRO, and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.

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