

How to Assemble

Step 1: Assemble Liner




- Attach self-aligning lid to liner, using moderate pressure.
- Adhesive on lid should adhere to entire circumference of the liner.

Step 2: Place liner in Outer Shell



- Place liner assembly in outer shell
- Connect blue vacuum tubing to vacuum port
- Turn valve to **ON** position

Step 3: Lid Set-up

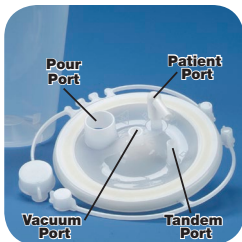

- Secure cap on Tandem and Pour Ports
- Place suction tubing on Patient Port (elbow optional)

Step 4: Liner Disposal



- Turn vacuum source **OFF** using the **ON/OFF** valve on the outer shell
- Disconnect tubing from patient port
- Remove blue hose from VACUUM port
- Securely cap all ports
- Remove liner leaving outer shell in place*
- Dispose of drainage and disposable liner in compliance with facility's guidelines*

***Do not dispose of outer shell when replacing the inner liner**

This Assembly Guide does not supersede the instructions given by a prescriber or the information contained in the Instructions for Use ("IFU") which accompany this device. Please refer to the IFU for all warnings, cautions and complete instructions.



Improving Care. Improving Business.®