



Improving Care. Improving Business.®

STEEP-T™ MULTI SPECIALTY PATIENT POSITIONING SYSTEM

	DO NOT REUSE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product.

INTENDED USE

The STEEP-T™ Multi Specialty Patient Positioning System is intended to help prevent unwanted movement when patients are in steep-angled positions while also protecting patients from nerve injuries, potential pressure injuries, and/or tissue breakdown.

KIT CONTENTS

M60-SF501	Foam pad w/handles (M60-SF500) Head positioner Scapula wedges Arm wedges (M60-SF600) Chest strap
M60-SF502	Foam pad w/handles (M60-SF500) Scapula wedges Arm wedges (M60-SF600) Chest strap
M60-SF503	Foam pad w/handles (M60-SF500) Chest strap
M60-SF510F	Flat anti-skid O.R. mattress pad
M60-SF520A	Angled anti-skid O.R. mattress pad

WARNINGS

- Prior to use, inspect the kit components for damaged, missing, or contaminated product.
- **DO NOT** attempt to carry patient while using this device.
- Use only as directed.
- Ensure kit components are installed correctly and patient is properly positioned on the device to prevent movement.
- **DO NOT** use kit without all components included in kit.
- Discontinue use if reaction occurs.

CAUTIONS

- The components within this system have only been tested in conjunction with each other. Use of outside components could alter proper



performance.

- Ensure the O.R. table pad is attached securely to the O.R. table and is clean and free of residue.
- Be sure to follow your hospital's policies and procedures for patient monitoring.
- **DO NOT** use on patients over 500lbs.
- Discontinue use if device does not properly support patient.

DIRECTIONS FOR USE

1. Place the foam pad with handles on the O.R. table ensuring that the non-skid surface interlocks with the non-skid surface of the O.R. mattress pad.
2. Once patient has been transferred onto the O.R. table the foam pad can be utilized, using the attached handles to reposition the patient as needed for ease of intubation and into final surgical position. **NOTE:** When repositioning the foam pad ensure the pad is completely lifted off the interlocking O.R. table pad. The non-skid materials are designed to not slide on one another.
3. Once patient is properly positioned, place the scapular wedges under the shoulder for additional points of fixation by using one hand to raise the patient's shoulder and placing a scapular wedge slightly angled toward the sternum or the midline of the body.
4. To place the arm wedge, gently roll the patient toward the clinician and position the wedge portion under the patient with the peak of the wedge aligned with the lateral edge of the torso. Align the top of the device with the middle of the humerus or upper arm.
5. After one arm positioner is placed, place the arm in the device and close securely with the hook and loop closure system, taking care to ensure circulation is not compromised.
6. Repeat with the other arm.
7. Before procedure begins make a visual assessment to ensure the surgical team is in agreement with final patient positioning.



Improving Care. Improving Business.®

CLEANING INSTRUCTIONS

- Follow hospital protocol.
- DeRoyal has validated that the following cleaning agents or their equivalents are acceptable for cleaning:
 - Hydrogen Peroxide
 - Dispatch Hospital Cleaner Disinfectant

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufacturer:

DeRoyal Industries, Inc.

200 DeBusk Lane

Powell, TN 37849 USA

888.938.7828 or (001) 865.938.7828

www.deroyal.com

PART#0-2340 REVISED 12/19

©2019 DeRoyal Industries, Inc.

All Rights Reserved. STEEP-T, DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.