



# Cordless Monitor

Compatible with:

- Cordless Bed Pad M2200-CBP
- Cordless Chair Pad M2200-CCP



M2100-C

## INSTRUCTIONS FOR USE

### SETTING UP THE MONITOR AND SENSOR PAD

1. Open battery compartment door with a small Phillips head screwdriver. Insert batteries.
2. Close the battery door and secure it with the small Phillips head screwdriver.
3. Place cordless bed and/or chair pad(s) within reach. Monitors can accommodate up to two cordless, bed and/or chair pads (does not accommodate two chair pads).
4. Unsnap the flap on the neck of the pad and turn the transmitter switch to the on position. Close the flap and secure the closure.
5. Press the program button located on the back of the monitor and hold until the monitor beeps three times.
6. Put pressure on the sensor pad to activate the pad and fall monitor. NOTE: Use a flat, hard surface to activate the pad and monitor. The fall monitor will beep two times. Press the reset button to activate the system.
7. The “Pad 1” light will blink while pressure remains on the pad.
8. If the “Pad Signal Lost” light is on and the system does not test properly, reduce the distance between the fall monitor and sensor pad. The ideal distance between the monitor and sensor pad is 15 to 20 feet.
9. Test the system before use.

### TURNING OFF OR RESETTING THE ALARM

1. When the alarm is triggered, the alarm can be turned off and reset with either the reset button or Caregiver Key™ function.
2. Select either “Reset Button ON” or “Caregiver Key™ ON” with the switch on the side of the monitor. Use the end of a pen to move the switch.
3. If set on “Reset Button ON” position, the caregiver can reset the monitor by pressing the reset button located on the front of the monitor.
4. If set on the “Caregiver Key ON” position, the monitor can only be reset by using the Caregiver Key provided with the monitor. The Caregiver Key must be placed over the red dot to reset the monitor.

### SELECTING A MELODY ALARM TONE

1. To select a different melody alarm tone, open the battery compartment and move the switch to the desired setting. There are three melody options.
2. To adjust the volume, turn the volume dial located in the inside of the battery compartment.

### ACTIVATING TWO SENSOR PADS

If activating a second pad, activate the first pad as instructed. DO NOT PRESS THE RESET BUTTON. The “Pad 2” light will illuminate. Put pressure on the second pad and release. The fall monitor will beep two times again and the light will turn off. At this time, press the reset button and both pads will be activated.

### MOUNTING THE MONITOR

1. The strap with buckle can be used to attach the monitor to a desired location.
2. To mount the monitor, mount the screw onto the wall or headboard. Hang the monitor on the screw using the mounting hole on back of the monitor.

### CARE INSTRUCTIONS

1. Wipe clean with disinfectant wipes.
2. After cleaning, test device before each use.

### NOTE:

1. To disengage alarm when moving patient, place Caregiver Key over red dot marked “Caregiver Key”. When patient re-engages alarm sensor, remove Caregiver Key to activate.
2. The Caregiver Key can be used in both “Reset Mode” and “Caregiver Key Mode” to disengage alarm.

