

1. Skin Prep

- Wipe temporal area with alcohol wipe and allow to dry

2. Sensor Placement

- Place sensor on temporal artery
- Gently press for 2-3 seconds
- Avoid placing on the temple “soft spot” and sinuses
- Do not place on hair
- Avoid exposing to surgical prep solutions or other fluids
- Do not reposition

Helpful Tips

- ENT procedures - place sensor on non-operative side
- BIS™ monitoring - apply TTP sensor prior to applying BIS™ sensor
- Prone procedures – place sensor so that tab is toward the top of the head (reverse of image)

