

Bag Canister

Setup Guide



Place the outer shell in the bracket on the roll stand mount in an upright position.



Cut the silicone tube to the desired length and connect it to the vacuum connector on the canister.



Open the suction liner and place it in the outer shell. Do not fully unfold the suction liner.



Turn on the vacuum and use your finger to cover the patient port. This will allow the bag to expand and fill the outer shell.



Connect the suction tubing to the patient port and begin the procedure.



After the procedure is completed, leave the vacuum on while removing the patient tubing and connector. Close the patient port using the patient port cap.



Turn off the vacuum and remove the suction liner by using the lift handles on the lid. Dispose of the suction liner according to facility protocol.

This application guide does not supersede the Instructions for Use (IFU) which accompany this device. Please refer to the IFU for all warnings, cautions and complete instructions.

The DeRoyal logo is a registered trademark of DeRoyal Industries, Inc.
DeRoyal Industries, Inc. | 200 DeBusk Lane | Powell, TN 37849 | 1.888.938.7828 | DeRoyal.com
© 2026, DeRoyal. All Rights Reserved. | Rev. 1/26 | Reprint #0-2651