

DeRoyal Wrist Splint

Single Patient Use Only

Rx Only

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use.

INTENDED USE


The splint is used for wrist support, Carpal Tunnel Syndrome, tendonitis and sprains and strains of the wrist.

PRECAUTIONS

Consult your clinician immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product.

DIRECTIONS FOR USE


1. Open all hook-and-loop closures, slide hand through the splint sleeve.
2. Position splint with palmar side of the splint below the palmar crease. Determine if metal palmar stay offers proper positioning.
3. Remove splint if wrist positioning needs to be adjusted. Adjust metal stay for proper positioning and fit. Reinsert in wrist pocket.
4. Reapply splint with palmar side of the splint below the palmar crease.

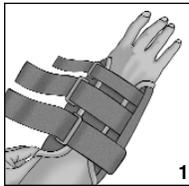

5. Comfortably secure the wrist straps through the d-ring closures to hold splint in proper position.



6. Trim excess strapping where needed.

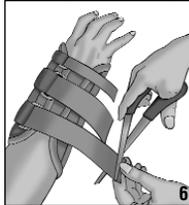

CLEANING INSTRUCTIONS

Hand wash in cold water using mild detergent.




Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Authorized Representative
DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.80100

Part #0-871
Revised 3/03
©2003 DeRoyal. All Rights Reserved.

Revision A

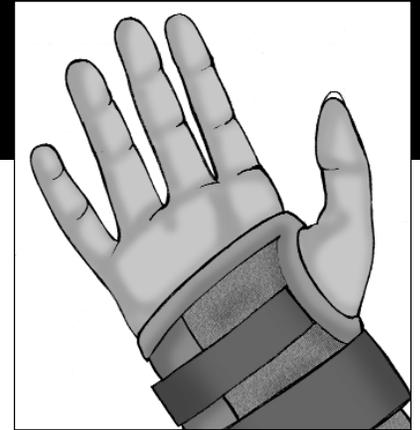

DeRoyal Wrist Splint

Single Patient Use Only

Rx Only

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use.

INTENDED USE


The splint is used for wrist support, Carpal Tunnel Syndrome, tendonitis and sprains and strains of the wrist.

PRECAUTIONS


Consult your clinician immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product.

DIRECTIONS FOR USE


1. Open all hook-and-loop closures, slide hand through the splint sleeve.
2. Position splint with palmar side of the splint below the palmar crease. Determine if metal palmar stay offers proper positioning.
3. Remove splint if wrist positioning needs to be adjusted. Adjust metal stay for proper positioning and fit. Reinsert in wrist pocket.
4. Reapply splint with palmar side of the splint below the palmar crease.

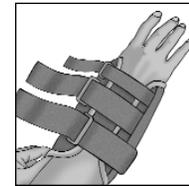

5. Comfortably secure the wrist straps through the d-ring closures to hold splint in proper position.

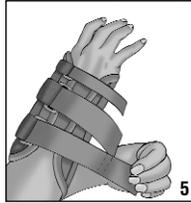

6. Trim excess strapping where needed.

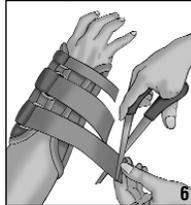

CLEANING INSTRUCTIONS

Hand wash in cold water using mild detergent.



Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Authorized Representative
DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.80100

Part #0-871
Revised 3/03
©2003 DeRoyal. All Rights Reserved.

Revision A