**Explanation of Symbols****Explicación de los símbolos****Explication des symboles****Erläuterung der Symbole****Spiegazione dei simboli****Explicação dos Símbolos****Verklaring van de symbolen****Symbolförklaringar****Условные обозначения****Επεξήγηση των συμβόλων****Rx Only**

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DeRoyal®

Improving Care. Improving Business.

Manufacturer:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
988.938.7828 or 865.938.7828  
www.derooyal.com  
Distributed in the US & Worldwide  
by DeRoyal Industries, Inc.

EC REP QNET BV  
Hommertweg 286  
6436 AM Amstenrade  
The Netherlands

Part #0-FULLSHELLWALKER2CEIFU  
Revised 9-2014

©2014 DeRoyal Industries, Inc.  
All Rights Reserved.

9/2014

**FULL SHELL WALKER 2**

**RX ONLY**  
**SINGLE PATIENT USE**  
**NON-STERILE**  
**NOT MADE WITH NATURAL RUBBER LATEX**

**IMPORTANT INFORMATION**

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

**INTENDED USE**

The DeRoyal® Full Shell Walker 2 is designed for grade 2 and 3 ankle sprains; stable fractures of tibia, fibula, talus and tarsus; post-operative immobilization (soft tissue reconstructions and open reduction internal fixation); Metatarsal fractures.

**CONTRAINDICATIONS**

Patients with poor peripheral circulation, diabetes or decreased skin sensitivity should not use this product.

**PRECAUTIONS**

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for duration of use. These instructions do not supersede hospital protocol and/or direct orders of patient's physician. Discontinue use of the device if it becomes damaged in any way, limiting its ability to perform as intended.
- Use only as directed by your medical professional.

**INSTRUCTIONS FOR USE**

- Unfasten the hook and loop straps, remove anterior panel and open liner.
- While seated, place leg in brace and position heel at the back of the brace. Wrap liner around the leg and foot. Secure with hook and loop closure.
- Replace the anterior panel.
- Fasten the hook and loop straps from the bottom to the top.
- Adjust airbladder compression by pushing the bulbs on either side of the boot. To deflate the air bladders, open the cap of the smaller button next to the bulbs and push the button to release the air. (See Image 1 & 2)

- Extra softgoods are included. (See Image 3)
- Adjustable, optional toe guard for additional protection. (See Image 4)

Application videos are available on [www.DeRoyal.com](http://www.DeRoyal.com).

**CLEANING AND/OR MAINTENANCE**

Remove soft goods and wash with mild detergent in lukewarm water. Air dry.

**WARRANTY**

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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