




## Algidex Ag® Packing Gauze

Single Use Only  
Disposable • Sterile  
**Rx Only**

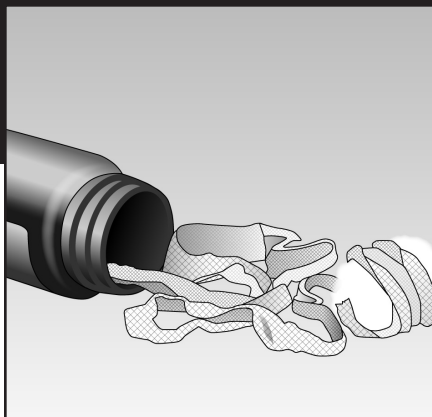

### IMPORTANT

Read all instructions carefully and completely before using this product.

### INTENDED USE

Algidex Ag® is an antimicrobial, hydrophilic wound dressing intended to manage wound exudate.

### INDICATIONS

Indicated for wounds that require packing as directed by a physician, such as:

- Pressure ulcers, stages I – IV
- Stasis ulcers
- Dermal wounds
- Venous ulcers
- 1st and 2nd degree burns
- Donor sites

### CONTRAINDICATIONS

Do not use on:

- Third degree burns.
- Ulcers resulting from infections.
- Lesions in patients with active vasculitis.
- Patients with sensitivity to silver, silver compounds, or alginates.

### WARNINGS

- Do not use when contraindicated.
- Do not use if product is expired or if packaging is open or damaged.
- Discard excess product after opening for initial use.
- This product is prescription only. Do not provide it to the patient without providing complete use instructions.

### DIRECTIONS FOR USE

1. Always wear gloves when dealing with open wounds.
2. Thoroughly cleanse the wound with saline.
3. Completely fill the wound with gauze, packing it lightly.
4. Cover the wound with the appropriate secondary dressing.
5. Change as desired, at a minimum every five days.
6. To change, remove the product gently.
7. Remove any product residue with saline, if desired.

Store this product in a cool, dry place. Dispose of this product in accordance with local, state, and federal guidelines.

### WARRANTY

DEROYAL PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



**Manufacturer**  
DeRoyal Industries, Inc.  
200 DeBusk Lane, Powell, TN 37849 U.S.A.  
888.938.7828/865.938.7828


©2015 DeRoyal. All Rights Reserved.  
Part #810880 Revised 8/15

Algidex Ag® is a registered trademark of DeRoyal Industries, Inc.



## Algidex Ag® Packing Gauze

Single Use Only  
Disposable • Sterile  
**Rx Only**

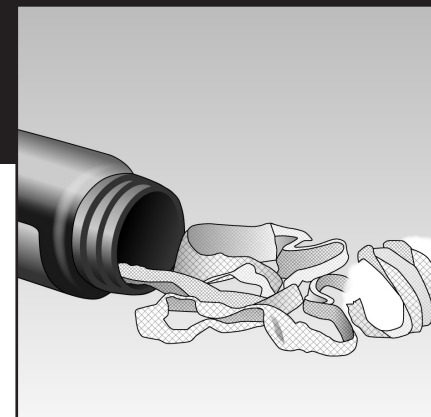

### IMPORTANT

Read all instructions carefully and completely before using this product.

### INTENDED USE

Algidex Ag® is an antimicrobial, hydrophilic wound dressing intended to manage wound exudate.

### INDICATIONS

Indicated for wounds that require packing as directed by a physician, such as:

- Pressure ulcers, stages I – IV
- Stasis ulcers
- Dermal wounds
- Venous ulcers
- 1st and 2nd degree burns
- Donor sites

### CONTRAINDICATIONS

Do not use on:

- Third degree burns.
- Ulcers resulting from infections.
- Lesions in patients with active vasculitis.
- Patients with sensitivity to silver, silver compounds, or alginates.

### WARNINGS

- Do not use when contraindicated.
- Do not use if product is expired or if packaging is open or damaged.
- Discard excess product after opening for initial use.
- This product is prescription only. Do not provide it to the patient without providing complete use instructions.

### DIRECTIONS FOR USE

1. Always wear gloves when dealing with open wounds.
2. Thoroughly cleanse the wound with saline.
3. Completely fill the wound with gauze, packing it lightly.
4. Cover the wound with the appropriate secondary dressing.
5. Change as desired, at a minimum every five days.
6. To change, remove the product gently.
7. Remove any product residue with saline, if desired.

Store this product in a cool, dry place. Dispose of this product in accordance with local, state, and federal guidelines.

### WARRANTY

DEROYAL PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



**Manufacturer**  
DeRoyal Industries, Inc.  
200 DeBusk Lane, Powell, TN 37849 U.S.A.  
888.938.7828/865.938.7828

©2015 DeRoyal. All Rights Reserved.  
Part #810880 Revised 8/15

Algidex Ag® is a registered trademark of DeRoyal Industries, Inc.