**SHARPSTOP™ SHARPS TRANSFER DEVICE**

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PART#4-10703G | REVISED 10/23

©2023 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Sharp Stop are trademarks or registered trademarks of DeRoyal Industries, Inc.

SHARPSTOP™ SHARPS TRANSFER DEVICE

	DO NOT REUSE
	MEDICAL DEVICE
	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct use is essential for the proper functioning of the product. This product is for use only as intended by healthcare professionals.

INTENDED USE

Sharpstop™ Sharps Transfer Device is a sterile, disposable device engineered to be used to transfer and pass a wide range of sharps instruments between members of the scrub team during any surgical or other invasive procedure.

INDICATIONS

Use where engineered controls are required to reduce employee exposure to bloodborne pathogens. Typically should be used during any invasive procedure using instruments which create a hazard for occupational injuries. This transfer device is designed to help healthcare facilities comply with OSHA directives.

WARNINGS

- **DO NOT** attempt to use both sides of the tray simultaneously.
- **DO NOT** transfer tray while sharps devices are not appropriately located within the device. **DO NOT** overfill the device.
- Discard the device(s) after use. **DO NOT REUSE. DO NOT** re-sterilize.
- Never reach blindly into the device.
- Use care when placing instruments into the device.
- Always visualize the instrument when removing or adding from the device.
- **DO NOT** use if packaging is open or damaged. Visually inspect product for damage prior to use.

CAUTIONS

- Always use caution when handling any sharps device.

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- Open package using sterile technique.
- Place the device on a stable surface within the sterile field, or hold using grips on the side or underneath the device.

DIRECTIONS FOR USE

NOTE: This transfer device accommodates handle sizes 3, 4, 5, 7 and 9 loaded with scalpel blade sizes 10, 11, 12, 15, 20, 22 and 23. **DO NOT** place more than one scalpel in the device at a time.

1. Turn the transfer device until the side containing two sloping ramps and the central finger guides is facing up.
2. To place a scalpel in transfer device for loading or for return after use, carefully place the scalpel into the device with the handle directed to one side of the ramp and the scalpel tip to the other side of the ramp.
3. To remove a scalpel from the transfer device, insert fingers into center circular recess on either side of scalpel to grasp the handle.
4. Either side of the transfer device may be used to transfer other sharps devices if they can be safely removed.

STORAGE CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal products are warranted for one hundred and twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

ADVERTISING

- **DO NOT** attempt to use both sides of the tray simultaneously.
- **DO NOT** transfer tray while sharps devices are not appropriately located within the device. **DO NOT** overfill the device.
- Discard the device(s) after use. **DO NOT REUSE. DO NOT** re-sterilize.
- Never reach blindly into the device.
- Use care when placing instruments into the device.
- Always visualize the instrument when removing or adding from the device.
- **DO NOT** use if packaging is open or damaged. Visually inspect product for damage prior to use.

CAUTIONS

- Always use caution when handling any sharps device.

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Ciddi olaylar, hastanın ikamet ettiği ülkedeki yetkili makama ek olarak DeRoyal Industries, Inc.'ye bildirilmelidir.