
## EN: SAFELINER® SEMI-RIGID LINER SYSTEM

MANUFACTURED BY:

DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.deroval.com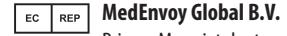

MedEnvoy Global B.V.  
Prinses Margrietplantsoen 33 – Suite 123  
2595 AM The Hague  
The Netherlands

UK Responsible Person  
MedEnvoy UK Limited  
85, Great Portland Street, First Floor  
London, W1W 7LT  
United Kingdom

PART #786293G I REVISED 10/23

©2023 DeRoyal Industries, Inc.  
All Rights Reserved. DeRoyal, the DeRoyal logo, and SafeLiner are registered trademarks of DeRoyal Industries, Inc.

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## SAFELINER® SEMI-RIGID LINER SYSTEM

LINERS:

|         |  |
|---------|--|
|         | DO NOT REUSE   |
|         | MEDICAL DEVICE   |
|         | NON-STERILE  |
|         | NOT MADE WITH NATURAL RUBBER LATEX   |
| RX ONLY | FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER. |

OUTER SHELLS:

|         |  |
|---------|--|
|         | NON-STERILE  |
|         | NOT MADE WITH NATURAL RUBBER LATEX   |
| RX ONLY | FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER. |

### IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of product.

### INTENDED USE

The DeRoyal® SafeLiner® Semi-Rigid Liner System is intended to aid in the management and disposal of waste fluid during a variety of medical procedures.

### CONTRAINDICATIONS

- NOT intended for long-term storage of bodily fluids.
- Fluids that may enter the system should not be reintroduced into the body.
- NOT recommended for pleural drainage.

### WARNINGS

- Product vacuum resistance tested to 27 inHg (685mmHg). DO NOT EXCEED THIS PRESSURE.
- Minimum recommended pressure: 30 mmHg.
- Inspect the outer shell assembly for visible cracks or broken components. The rim should be smooth and free of any burrs to ensure proper function. Replace at healthcare professional's discretion.
- DO NOT use lid if it is or has been wet as product will not allow suction.
- DO NOT reuse liner or attempt to sterilize the outer shell.
- DO NOT interchange liners and outer shells with any other brand. Liners must be used with appropriately sized DeRoyal outer shell only. Different sizes may result in non-compliance with facility guidelines.
- DO NOT adhere the lid to or onto the outer shell of canister. A liner must always be used. Only insert assembled liner lid and liner base into outer shell for correct utilization.
- Outer shell is reusable in certain patient care settings. (See Cleaning Instructions.)

### DIRECTIONS FOR DISPOSABLE SUCTION LINER

- Assemble the lid to liner base. Ensure adhesive on lid adheres to entire diameter of liner base.
- Place the liner assembly onto outer shell so that the lid is seated around the entire ridge of the outer shell.
- Attach blue vacuum tubing from neck of outer shell to the "VACUUM" port on liner.
- Attach patient tubing to "PATIENT" port.
- For single canister use, securely cap "TANDEM" port and pour spout.
- For tandem canister use, continue with the instructions below.
- Check to make sure all caps tethered to sides of lid are free and not trapped between the lid and liner.

### Instructions:

#### CAUTIONS

- Ensure all connections are secure. Improper set up may result in product malfunction or loss of suction.
- Avoid lifting, which can result in premature shutdown or inaccurate volumetric readings. The product should be maintained on a level surface.

- Induce vacuum into canister only at the beginning of procedure. Shut off vacuum when not in use. For optimal performance, it is recommended that the suction tubing used is no less than 6mm (~1/4") in diameter and no longer than 6 meters (~20 feet) in length; however, tubing with a 3/16" internal diameter may be used.
- All measurements are approximate.

- When handling the SafeLiner® canister, utilize universal personal protective equipment including gloves, face mask, and proper clothing to ensure protection from splashes during transport and disposal of aspirated fluid in the container.

### CLEANING INSTRUCTIONS FOR OUTER SHELL

- If outer shell becomes dirty, cleanse with appropriate cleaning agents.
- Various household cleaners are appropriate. A 10% bleach solution or 10% alcohol solution will clean the outer shells without damaging the plastic.
- Acetone-based cleaners should be avoided as they may damage the shell.

### STORAGE AND TRANSPORT CONDITIONS

DON'T store product at extreme temperatures or in a moist damp environment.

|  |                         |
|--|-------------------------|
|  | KEEP DRY                |
|  | KEEP AWAY FROM SUNLIGHT |

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### DIRECTIONS FOR OUTER SHELL

- Place outer shell(s) in hardware bracket on the wall or roll stand. For direct-to-wall connection, shells, threads & diameter index safety device (DIS) connect directly onto the regular vacuum port.
- On wall-mounted shells, connect one end of tubing from the regulator on the wall to the valve on the side of the bracket.
- The blue vacuum tubing should be connected to the outer shell of the bracket.

3. The blue vacuum tubing should be connected to the outer shell of the bracket.

4. Connect the blue vacuum tubing to the "VACUUM" port on the liner.

5. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

6. Disconnect patient tubing from the "PATIENT" port of the first liner.

7. Remove the blue hose(s) from the "VACUUM" ports.

8. Disconnect patient tubing from the "PATIENT" port of the second liner.

9. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

10. Disconnect patient tubing from the "PATIENT" port of the third liner.

11. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

12. Disconnect patient tubing from the "PATIENT" port of the fourth liner.

13. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

14. Disconnect patient tubing from the "PATIENT" port of the fifth liner.

15. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

16. Disconnect patient tubing from the "PATIENT" port of the sixth liner.

17. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

18. Disconnect patient tubing from the "PATIENT" port of the seventh liner.

19. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

20. Disconnect patient tubing from the "PATIENT" port of the eighth liner.

21. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

22. Disconnect patient tubing from the "PATIENT" port of the ninth liner.

23. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

24. Disconnect patient tubing from the "PATIENT" port of the tenth liner.

25. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

26. Disconnect patient tubing from the "PATIENT" port of the eleventh liner.

27. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

28. Disconnect patient tubing from the "PATIENT" port of the twelfth liner.

29. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

30. Disconnect patient tubing from the "PATIENT" port of the thirteenth liner.

31. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

32. Disconnect patient tubing from the "PATIENT" port of the fourteenth liner.

33. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

34. Disconnect patient tubing from the "PATIENT" port of the fifteenth liner.

35. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

36. Disconnect patient tubing from the "PATIENT" port of the sixteenth liner.

37. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

38. Disconnect patient tubing from the "PATIENT" port of the seventeenth liner.

39. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

40. Disconnect patient tubing from the "PATIENT" port of the eighteenth liner.

41. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

42. Disconnect patient tubing from the "PATIENT" port of the nineteenth liner.

43. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

44. Disconnect patient tubing from the "PATIENT" port of the twentieth liner.

45. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

46. Disconnect patient tubing from the "PATIENT" port of the twenty-first liner.

47. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

48. Disconnect patient tubing from the "PATIENT" port of the twenty-second liner.

49. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

50. Disconnect patient tubing from the "PATIENT" port of the twenty-third liner.

51. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

52. Disconnect patient tubing from the "PATIENT" port of the twenty-fourth liner.

53. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

54. Disconnect patient tubing from the "PATIENT" port of the twenty-fifth liner.

55. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

56. Disconnect patient tubing from the "PATIENT" port of the twenty-sixth liner.

57. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

58. Disconnect patient tubing from the "PATIENT" port of the twenty-seventh liner.

59. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

60. Disconnect patient tubing from the "PATIENT" port of the twenty-eighth liner.

61. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

62. Disconnect patient tubing from the "PATIENT" port of the twenty-ninth liner.

63. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

64. Disconnect patient tubing from the "PATIENT" port of the twentieth liner.

65. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

66. Disconnect patient tubing from the "PATIENT" port of the twenty-first liner.

67. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

68. Disconnect patient tubing from the "PATIENT" port of the twenty-second liner.

69. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

70. Disconnect patient tubing from the "PATIENT" port of the twenty-third liner.

71. Turn on the vacuum source to each liner using the ON/OFF valve on the outer shells.

72. Disconnect patient tubing from the "PATIENT" port of the twenty-fourth liner.

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