

XTW[®]

Pediatric CERVICAL COLLAR

SIZING GUIDE

INFANT SHORT	1 - 18 months
	Weight: 11 - 29 lbs. Height: 21 - 33 in.
INFANT REG	9 - 24 months
	Weight: 22 - 33 lbs. Height: 29 - 37 in.
CHILD TOT	1 - 3 years
	Weight: 24 - 36 lbs. Height: 33 - 40 in.
CHILD SHORT	2 - 5 years
	Weight: 26 - 42 lbs. Height: 35 - 45 in.
CHILD REG	3 - 6 years
	Weight: 27 - 54 lbs. Height: 37 - 48 in.

 DeRoyal[®]
(888) 938-7828
www.deroyal.com

XTW[®]

CERVICAL COLLAR

SIZING GUIDE



Chin
A
Trapezius

Set the patient's head in a neutral position. Place the bottom of the sizing guide at the top of the trapezius and flat against the patient's head. An imaginary line should be drawn from the chin back to the guide. The size should then be selected. In order to ensure a proper fit, the collar should not touch the tracheal area and the patient's chin should be flush against the chin piece.

 DeRoyal[®]
(888) 938-7828
www.deroyal.com

Extra Tall ($\geq 1 \frac{7}{8}$ ")

Tall ($1 \frac{1}{4}$ " - $1 \frac{7}{8}$ ")

Regular ($\frac{5}{8}$ " - $1 \frac{1}{4}$ ")

Short ($\frac{1}{4}$ " - $\frac{5}{8}$ ")

Extra Short ($\leq \frac{1}{4}$ ")